# AESCULAP®

Inklusive Neuheiten  
New Products included

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**CRANIOFIX®** - Implant system for refixation of cranial bone flaps after craniotomy

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**CRANIOFIX®** has been developed in cooperation with:

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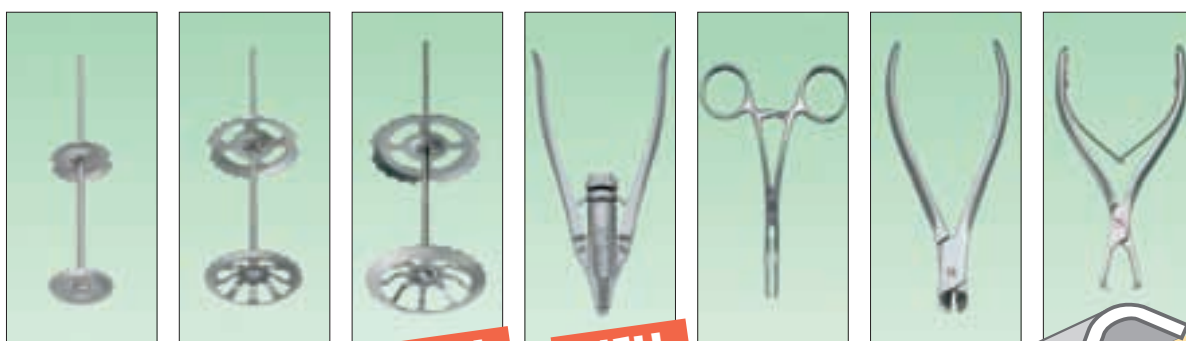
Bone flaps after craniotomy are typically fixed  
with wire, suture material or mini plates and  
screws.

In some cases, fixation with wire or suture  
material is not secure. Shifting of the bone flap  
may result in dislocation, causing depression or  
protrusion of the flap. This phenomenon  
occurs more frequently with the progressive  
shift to smaller craniotomies for minimally  
invasive surgical procedures.

Fixation of the bone flap using mini plates and  
screws has improved the attachment of the  
bone flap. This technique, however, demands a  
considerable amount of time and added cost.


This has led to the development of **CRANIO-  
FIX®** - a totally new, unique system for fixation  
of the bone flap to the cranium.

A distinguishing feature of **CRANIOFIX®** is its  
quick and easy application. It provides optimal  
stability and reliable fixation of the bone flap to  
the cranium.

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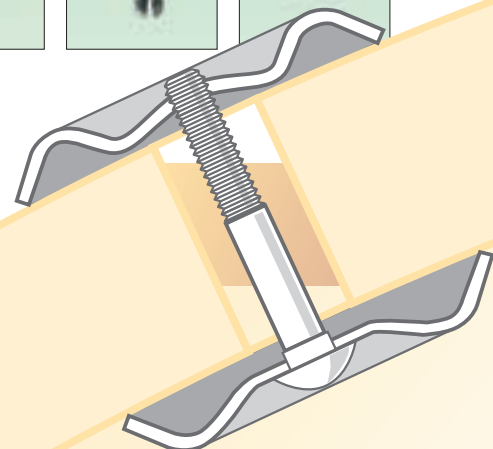### The **CRANIOFIX**® Titanium Clamps

- with 11 mm, 16 mm and 20 mm diameter of disks for positioning in craniotomy gap and burr hole
- press bone flap and cranium between the two disks
- the **CRANIOFIX**® titanium clamps with 16 mm and 20 mm disk diameter cover simultaneously the burr hole
- made of titanium alloy TiAl6V4
- biocompatible
- minimal artifacts in CT / MR
- MR-compatible

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### The Applying Forceps

- allow fast and accurate application of the **CRANIOFIX**® titanium clamps
- with automatic strain relief
- able to be dismantled

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#### The Holding Forceps

- for temporarily securing of the **CRANIOFIX**<sup>®</sup> titanium clamps
- prevent pressure of the **CRANIOFIX**<sup>®</sup> titanium clamp on the dura during application

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#### The Pin Cutter

- for simple removal of the protruding portion of the pin on the **CRANIOFIX**<sup>®</sup> titanium clamps
- blades are recessed so the protruding pin is cut off directly above the upper disk

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#### The Removal Forceps

- for simple removal of the **CRANIOFIX**<sup>®</sup> titanium clamps
- two teeth fit precisely into notches on the upper disk of the **CRANIOFIX**<sup>®</sup> titanium clamps

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The application of **CRANIOFIX**<sup>®</sup> is described with a circular craniotomy with a 12 mm burr hole.

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The **CRANIOFIX**<sup>®</sup>-titanium clamp with 16 mm diameter is positioned in the burr hole; the **CRANIOFIX**<sup>®</sup>-titanium clamp with 11 mm diameter in the craniotomy gap. The **CRANIOFIX**<sup>®</sup>-titanium clamps should be positioned equidistant to one another. The lower disks are inserted between the Dura and the cranium.

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The bone flap is then placed in its original position.

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The holding forceps is placed on the first **CRANIOFIX**<sup>®</sup> titanium clamp and holds the clamp in place to prevent unintentional pressure on the dura during application.

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The applicator is brought into position above the pin of the **CRANIOFIX**<sup>®</sup> titanium clamp.

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The handles of the applying forceps are pressed together to move the upper disk downward along the circumferential grooves toward the lower disk. Automatic strain relief upon reaching the maximum absorbable force through the **CRANIOFIX**<sup>®</sup> titanium clamp ensures that both disks are applied in an optimum ratio to one another. The lower disk holds the lamina interna of the vault of the cranium and bone flap from the inside with its toothed concave side while the upper disk holds the lamina externa from the outside.

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**CRANIOFIX**<sup>®</sup> forms a perfect closure connecting the bone flap to the craniotomy edge. Any supra-sutures in the dura can be performed and fixed in place below the upper disk.

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The pin cutter is used to remove the remaining external protruding portion of the pin, which must be held.

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The same procedure is performed with all remaining **CRANIOFIX**<sup>®</sup> titanium clamps. The result is a firm, positionally stable, reliable closure of the bone flap to the cranium with a simultaneously covering of the burr hole.

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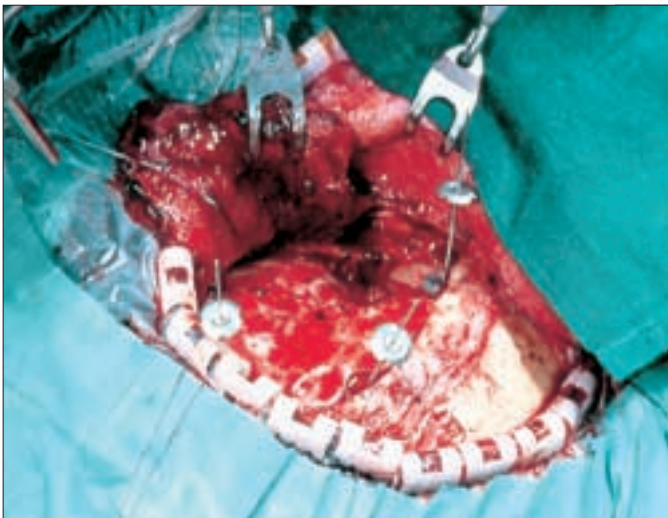
The case shown demonstrates the fixation of the bone flap in a left pterional craniotomy.

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
Three **CRANIOFIX**<sup>®</sup> titanium clamps are positioned equidistant to one another along the craniotomy opening. The lower disks are inserted between the dura and the cranium.

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
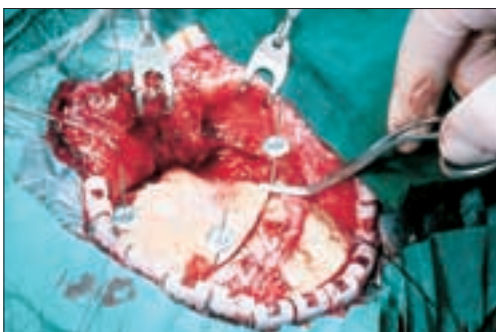
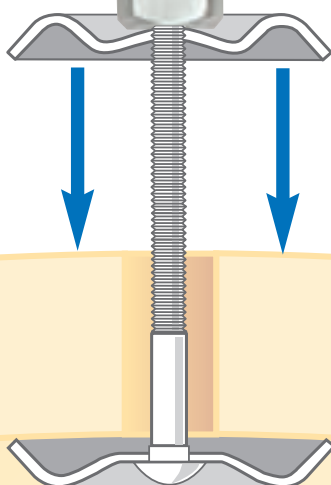
The handles of the applying forceps are pressed together to move the upper disk downward along the circumferential grooves toward the lower disk.

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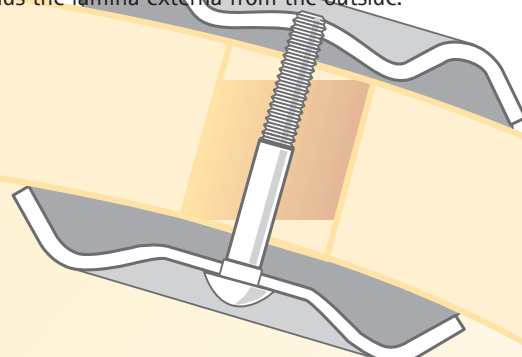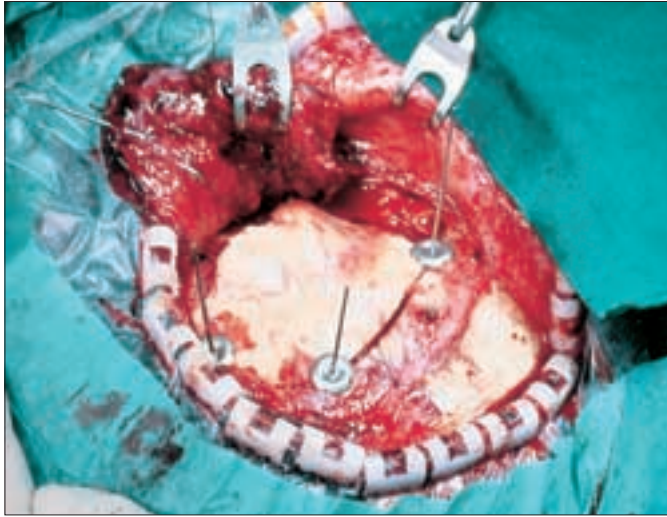
Automatic strain relief upon reaching the maximum absorbable force through the **CRANIOFIX**<sup>®</sup> titanium clamp ensures that both disks are applied in an optimum ratio to one another. The lower disk holds the lamina interna of the vault of the cranium and bone flap from the inside with its toothed concave side while the upper disk holds the lamina externa from the outside.

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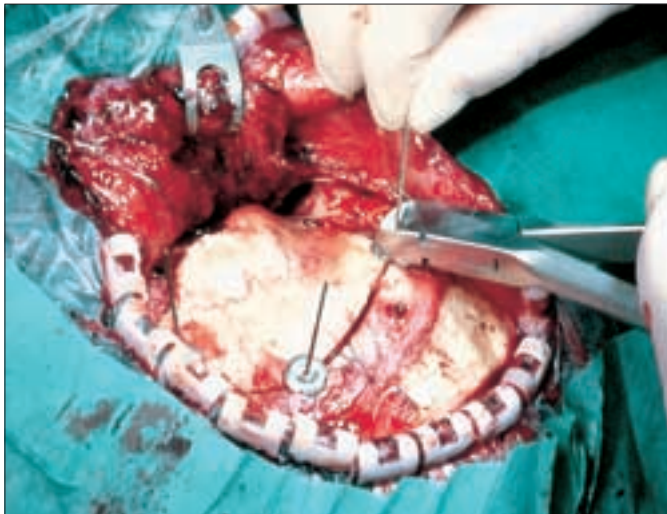
**CRANIOFIX®** forms a perfect closure connecting the bone flap to the craniotomy edge. Any supra-sutures in the dura can be performed and fixed in place below the upper disk.

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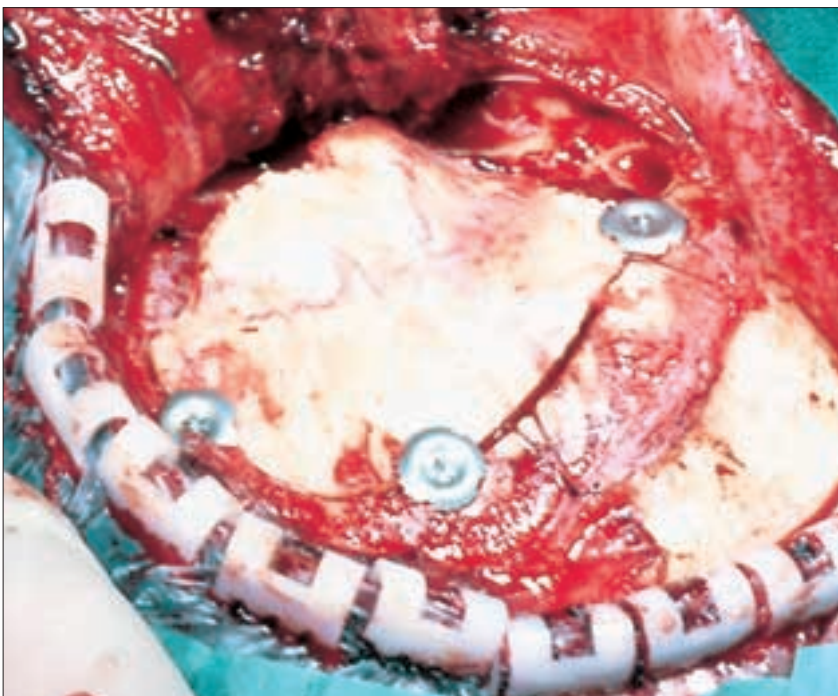The same procedure is performed with all remaining **CRANIOFIX®** titanium clamps. The result is a firm, positionally stable, reliable closure of the bone flap to the cranium.

## Postoperative Kontrolle Postoperative Follow-up Checks

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Post operative follow up x-rays show the stable fixation of the bone flap in the craniotomy opening and the precise form match to the curve of the cranial vault. (In this case, left pterional and right temporal.)

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Interpretation of CT and/or MR results are not adversely affected since titanium alloy produces minimal artifact.

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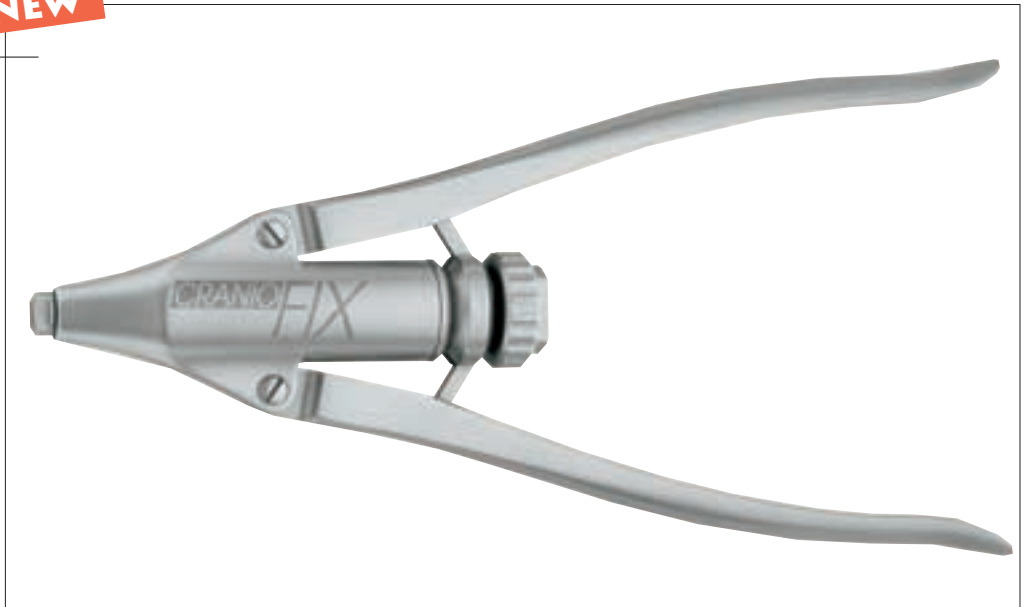
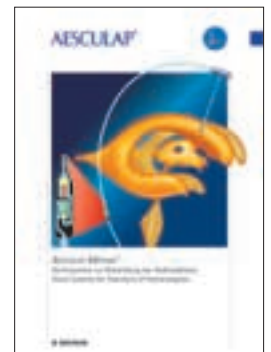
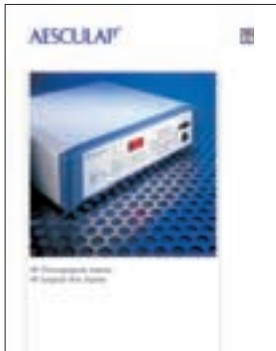
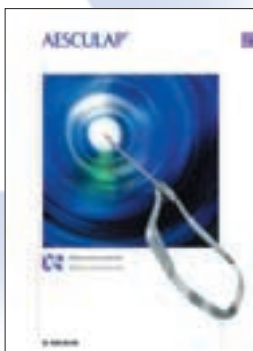
C 731 11 MRI-Instrumente  
MRI-Instruments


B 028 Lyostypt®  
Hämostyptikum  
Haemostatic Felt

C 770 11 XS  
Mikroinstrumente  
Micro Instruments

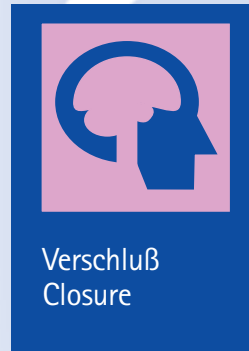C 745 11 SENSATION  
Mikroinstrumente  
Micro Instruments

## Verschluß Closure

B 055 NeuroPatch®  
Duraersatz  
Dura Substitute

B 076 Lyoplast®  
Duraersatz  
Dura Substitute

C 471 11 CranioFix®  
Knochendeckelfixierung  
Bone Flap Fixation

B 037 Premilene®  
Nahtmaterial

Safil® Synthetic Absorbable Suture

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All rights reserved. Technical alterations are possible.  
This leaflet may be used for no other purposes than offering, buying and selling of our  
products. No part may be copied or reproduced in any form. In the case of misuse we  
retain the rights to recall our catalogues and pricelists and to take legal actions.

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AESCULAP AG & CO. KG